Welcome

Welcome to the VetScan VUE, the first app-based addition to the VetScan line of products. With the VUE, you'l experience a new and innovative approach to VetScan Rapid Test diagnostics right from your mobile device. Simply download, launch, test, and share. Yes, it's that simple.

Let's get started

System Requirements

'ou must have access to the network nam and password in order to connect the VUE to your clinic's network via Wi-Fi or LAN. The VUE is currently compatible with 2.4 GHz wireless networks, and will not work on 5 GH frequency. All mobile devices must have Wi-Fi capability as well.

Your VUE is compatible with mobile devices using iOS 8.0 or later versions and Android devices using v.4.4 or later versions.

The VUE can only be used with the VUE App.

App Icons 🗯 🛛 🗭

- History: Browse patient results and share a PDF or XML file via email.
- Device: Connect to a current device or initiate a scan for a new device.
- Test: Perform a Quick Scan, Timed Scan, or Batch Scan.
- User: View or change your account information and password.
- Options: Configure app settings initiate a support ticket, and view disclaimers.
- Negative Result: The results have been determined negative for a particular test.
- **Positive Result:** The results have been determined positive for a particular test.
- Test Error: A test error has occurred. Follow the onscreen instructions or contact technica support.
- Share: Select results to share (PDF or XML) via email.

Unboxing

Carefully remove the VUE from its packaging and place on an even surface. Choose the appropriate AC adapter for your region and plue it into the micro USB port on the back of your VUE and to an approved power outlet.

Note: Please use only the power supply

provided with your VetScan VUE. The use of an incompatible power supply may cause damage to your device and void the warranty.

Press the power button down for 1 second to power on. The LED will flash RED first, then solid WHITE. It is now ready to onboard. Make sure vour mobile device is connected to your clinic's Wi-Fi before proceeding to App Setup.

AP VUEXXXXX AP password: XXXX-XXX

Note: Make note of the VUE AP SSID and AP password located on the bottom of the VUE. You will need this during the onboarding process.

App Setup

If you have not received an email from noreply-vetscanvue@abaxis.com by now, please contact us using the information in the Support & More section. Once you receive the email, open it in your browser on your phone and follow the instructions to set up your new password and download the app. If you are the clinic administrator, you can set up users for your clinic as well.

Once complete, launch the App. 🚺

You will be greeted with screens outlining the primary features of the VUE App. You can choose to skip these screens by pressing Skip (iOS) or the "x" (Android) in the top right-hand corner or review the screens to learn more about the features.

App Login

When the log-in screen appears, log in with the email and password you were assigned by Abaxis or the clinic administrator. Press LOGIN.

After logging in, you will be prompted to START DEVICE SETUP TO SCAN.

Select NEW DEVICE to onboard your device and connect to the clinic Wi-Fi. This will take you to the UNBOXING screen for further instructions. Press CONTINUE

To connect your VUE to your Android or iOS device, you can:

- 1. Follow the on-screen instructions
- 2 Visit
 - www.abaxis.com/veterinary/products/vetscan-vue 3. Access tutorials from the Help screen of the app

Testina

The VetScan VUE is designed for all Abaxis VetScan Rapid Tests. All tests must be performed according to their package insert before using the VUE. Sample and buffer must be added to the test before it is inserted into the VUE. Cassettes must be inserted with the sample well facing outwards. Please note that the cassette must be inserted into the VUE device before a scan can be initiated.Initiate the Rapid Test according to package insert.

Timed Scan:

1. Insert cassette into VUE immediately after adding the sample and buffer, press **O**Test Button, and press Timed Scan.

2. Press ASSIGN PATIENT to enter the ID.

3. When the timer has reached 00:00, the result will appear on the screen.

4. Read result and/or share or press X to exit the screen

Ouick Scan:

- 1. Insert cassette into VUE immediately after the test is completed according to the package insert.
- 2. Press 🚺 Test Button and press Quick Scan.
- 3. Press ASSIGN PATIENT to enter the patient ID.
- 4. Wait 5-10 seconds, then read result and/or share, or press X to exit the screen.

Batch Scan

The VetScan VUE is also designed to perform a batch scan. This allows you to perform multiple tests on a single patient in the time it takes to perform one test. It also eliminates the need to re-enter the patient ID in multiple times.

- 1. Initiate all tests according to the package insert for one specific patient.
- 2. Perform a Timed Scan for one of the tests.
- 3. Once complete, perform Quick Scans on the remaining tests, making sure to select the most recent Patient ID entered. This enables guick scanning without having to enter the Patient ID in multiple times for the same patient.

Results

The VetScan VUE is designed to interpret test results for all Abaxis VetScan Rapid Tests. However all results should be confirmed visually with the exception of the cPL Rapid Test, which must be interpreted by the VUE. Please refer to VetScan VUE Icons for more information on the result icons

Support & More

A product manual for your VetScan VUE is available

- To access the complete product manual and learn more about your VUE, please visit:
- www.abaxis.com/veterinary/products/vetscan-vue For Technical Support, available 24/7 including

holidays: In the Americas:

EMAIL: vetsupport@abaxis.com

TOLL FREE: +1-800-822-2947

PHONE: +1-510-675-6500

Outside the Americas:

- EMAIL: techsupport@abaxis.de
- PHONE: +49-6155-780-210

To place an order, contact your Abaxis Authorized Distributor or for questions, call Abaxis Customer Support at +1-800-822-2947 or email abvetcsreps@abaxis.com

Abaxis and VetScan are registered trademarks of Abaxis, Inc. The VetScan VUE is intended for veterinary use only. © Abaxis 2017

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Abaxis e VetScan são marcas registradas da Abaxis, Inc O VetScan VUE destina-se apenas para o uso
Worldwide Headquarters Abaxis, Inc. 3240 Whipple Road Union City, CA 94587 USA